## LIMISAN™

Painless adhesive bandage remover

For quick and easy removal of all adhesive bandages, tapes, and sticky residues

#### WHAT IS LIMISAN™?

Limisan™ is an easy-to-use spray for the immediate and painless removal of all adhesive bandages, including medical bandages, sports tapes, and patches, as well as sticky residues. Limisan™ uses a mild, sting-free formula, suitable for children and adults, which dries quickly, permitting the re-application of bandages without delay.

#### HOW DOES LIMISAN™ WORK?

Limisan™ contains a blend of ingredients that quickly form a layer between the adhesive and the skin. This layer interrupts the adhesive bond, causing a total loss of adhesion and allowing the adhesive bandage to be gently lifted away. All the ingredients in Limisan™ evaporate quickly, leaving no residue, and enable a new adhesive bandage to be applied immediately.

## HOW TO USE LIMISAN™ TO REMOVE ADHESIVE BANDAGES?

STEP 1 > Spray Limisan™ around the edges and on top of the adhesive bandage(s).

STEP 2 ➤ No waiting required. Peel the bandage away from the skin. Spray additional Limisan™, as needed.

NOTE: Limisan™ dries quickly and will not affect the adhesion of subsequent adhesive bandages.





# HOW TO USE LIMISAN™ TO REMOVE STICKY RESIDUES?

- 1. Sprav Limisan™ on a tissue.
- 2. Wipe the residue with the moistened tissue.

# HOW TO USE LIMISAN™ AROUND THE EYES?

- 1. Spray Limisan™ into the cap of a bottle or a shallow dish.
- 2. Dip a cotton swab in the product.
- Apply the wet cotton swab on the adhesive until it looks wet and gently remove the adhesive from the skin.

**NOTE:** Take care to avoid getting Limisan $^{\text{\tiny{M}}}$  into the eyes.

# **INGREDIENTS**

Disiloxane, butane, propane, isopropyl alcohol, bisabolol, spearmint.

# CONTENTS

34 g spray, 40 applications.

### **ADVANTAGES OF LIMISAN™**

- Pain free
- · Easy spray application
- No waiting, works in seconds
- · Dermatologically tested
- Non-irritant, no sting
- Suitable for children and adults
- Dries quickly, permits rapid re-application of bandage(s)

# **WARNINGS**

- For external use only.
- Limisan™ should not be used on a person who is allergic to any
  of the ingredients.
- Avoid getting the product into the eyes or other mucous membranes.
   In case of contact with eyes, rinse thoroughly with water.
- Do not swallow. If accidentally swallowed, do not induce vomiting, seek immediate medical attention.
- Do not inhale. Use only in well-ventilated areas. If accidentally inhaled, exposed individual should seek fresh air and rest in a comfortable position.
- Discontinue use in case of any undesirable effect and consult your doctor. Have the product and this leaflet at hand.
- · Keep out of reach of children.
- Container may explode if heated.
- Do not smoke while using this product.
- Do not use after the expiry date.

### **CAUTION**



## CONTAINER MAY EXPLODE IF HEATED.

Contents under pressure. Do not place in hot water or near radiators, stoves or other sources of heat. Do not puncture or incinerate container or store at temperatures over 50 °C.



### FLAMMABLE.

Do not use in presence of open flame or spark.

### **STORAGE**

- Store at room temperature in a well-ventilated place.
- · Protect from sunlight.

# FOR MORE INFORMATION, VISIT OUR WEBSITE: WWW.LIMISAN.CA

Imported by: SEARCHLIGHT

PHARMA

Montreal, Quebec H3J 1M1 www.searchlightpharma.com

1-855-331-0830